

CueMed Patient User Guide

Requirements:

- Apple iOS device (iPhone, iPad) with iOS version 15.6 or higher
- Ensure Bluetooth on your iOS device is enabled and Internet access is available on your device.
- Email address and password for CueMed application (provided by pharmacy)
- Download CueMed Mobile App ([App Store](#))
- CueSticker attached to a medication blister pack

Quick Start Guide:

1. Download and install the CueMed Mobile App using the link provided above.
2. Open the CueMed mobile application on your device
3. The pharmacy will provide credentials for you to sign in to the CueMed mobile application.
4. Sign in using your credentials.
5. Once logged in you will be presented with the Adherence Overview screen. From this screen you will be able to:

Icon	Function	Description
	View current adherence statistics and summary.	Displays a summary of the adherence data collected from the medication blister pack assigned to the logged in patients
	View all assigned blister packs.	Displays all the blister packs that have been assigned to the logged in patient. • To view blister pack data, click on selected pack to view the pack data and individual cavity open times.
	Share Report Progress.	Allows the user to share the adherence report of the logged in user via email. • Enter the email recipient in the caregiver email box • Click Send Report to push a current report.
	View notification and alerts history.	Displays the history of alerts and notifications to the logged in user. (i.e. missed medications, etc.)
	Add new Packs to monitor.	Allows the logged in user to add new packs to CueMed to monitor for adherence. • Press and hold the Pharmacy Only button for 2 seconds to initialize the CueMed sticker, and click on the icon to view detected nearby packs. • Select the sticker to add then click on Activate pack to attach the pack to your user profile.

In some instances, you may be requested to activate the stickers manually by your pharmacy. To activate the CueMed sticker, press the Pharmacy Only button for 2-3 seconds until the LED blinks. Your sticker is now activated.

If you require technical support or have questions regarding setup, activation, or synchronization, please contact [CuePath Support](#)